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Dear patient,
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The GenuTrain P3 is a medical device. It is a support for centering and securing the patella with an adjustable corrective strap.

Indications

- Knee joint irritation (patellofemoral pain syndrome, patellar chondropathy, chondromalacia patellae, patella tip syndrome / jumper's knee, runner's knee)
- Retropatellar osteoarthritis
- Patellar lateralization
- Patellar subluxation / patellar dislocation (post-traumatic)
- High-riding patella (patella alta)
- Post-operative care (e.g. following stabilization of the patella)
- Feeling of instability

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the professional*.

The benefits of this product are felt particularly during physical activity.

- Remove the support during longer rest phases.
- Once you have been prescribed the GenuTrain P3, only use it according to your indication and any additional instructions given by a professional*.
- Please consult the professional* or your physician first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the GenuTrain P3 – e.g. an increase in discomfort – stop using it and contact your physician immediately.
- Caution: When doffing the product, there is not sufficient support for the injury. In this case, avoid any stress on the affected leg or foot.
- The GenuTrain P3 knee support cannot mechanically prevent the kneecap from dislocating again (patellar luxation) as a result of external trauma.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means

of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. This aid should be donned and worn only after consulting your physician if you have one of the following medical conditions:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. In case of any raised scars with swelling, redness or excessive heat build-up
2. Varicose veins
3. Impaired sensation in the leg due to neurological abnormalities
4. Impaired circulation in the leg (e.g. diabetes mellitus)
5. Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Donning of the GenuTrain P3

- ① Grip the support by the donning aids at the top ends of the lateral stays on both sides ② and pull the support up over the knee. ③ Position the ring-shaped pad in such a way that it encloses the kneecap without exerting pressure. ④ If required, correct the fit of the support.

Securing the kneecap in the desired position

- ⑤ To ensure that the kneecap is in the perfect position, bend the leg slightly (max. 20°). Release the tensioning strap of the corrective strap from the Velcro area on the inside of the support. Pull the strap diagonally upward until your kneecap is in the desired position that is comfortable for you. ⑥ Refasten the end of the strap onto the Velcro area.

Doffing of the GenuTrain P3

- ⑦ Grip the support by the aid for taking off your stockings at the bottom ends of the stays on both sides and pull the support downwards. Make sure not to turn the support inside-out.

Cleaning instructions

Secure the end of the corrective strap in the indicated position. Machine-wash your product at 30 °. Please use mild detergent and a mesh bag. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees ideal effectiveness of the product.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Part of the body this product is used for
see Indications and Intended purpose

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The product is supplied ready for use

Technical specifications / parameters

The GenuTrain P3 consists of a knitted section with an integrated massage pad around the kneecap, a corrective strap on top of this and an oval massage pad at the upper edge of the support. Spare parts are not available.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain P3 have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional*. Before first using our medical device, please seek the advice of a physician or professional* as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional* as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional*, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Silicone (SI), Thermoplastic polyurethane (TPU), Elastane (EL), Thermoplastic elastomer (TPE-SEPS), Polyurethane (PUR), Cotton (CO), Polydimethylsiloxane-polyurea block copolymer (based on PDMS-urea copolymer), Polyester (PES)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

* A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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