30°C
86°F

BAUERFEIND AG
 Triebeser Straße 16
 07937 Zeulenroda-Triebes
 Germany
 P +49 (0) 36628 66-40 00
 F +49 (0) 36628 66-44 99
 E info@bauerfeind.com
BAUERFEIND.COM

EpiPoint®

Ellenbogenspange
 Elbow strap
 Orthèse bracelet de coude
 Fibbia per il gomito
 Pasador para el codo

**MADE
 IN
 GERMANY**

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Polyamid (PA), Thermoplastisches Polyurethan (TPU), Styren-Ethylen / Butylen-Styren-Blockcopolymer (TPS-SEBS), Polyurethan (PUR), Polyoxymethylen (POM), Baumwolle (CO), Polyester (PES),

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 english

Dear Customer,

thank you for choosing a
Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended use

EpiPoint is a medical device. It is an orthosis¹ for the targeted relief of the tendon insertions at the elbow.

Indications

- Epicondylitis lateralis humeri (inflammation of the muscle origin, tennis elbow)
- Epicondylitis medialis humeri (inflammation of the muscle origin, golfer's and thrower's elbow)
- (Relapse) prevention

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist².

• **EpiPoint is a prescribed product that should be worn under a physician's guidance. EpiPoint should only be applied in accordance with these instructions for use and for the listed areas of application.**

- If you notice any unusual changes or your symptoms get worse, take the product off and please contact your physician without delay.
- No product liability is accepted in the event of improper use.
- The EpiPoint should not be altered arbitrarily.
- In the event of a product defect or wear to individual parts, please contact your medical retailer.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- The product's unique effect is produced primarily during physical activity. The product should be removed during prolonged periods of rest (e.g. sleeping, continuous sitting).
- Any supports and orthoses applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.
- In such cases, please loosen the straps – or the fastening slightly and have the correct fit and size of the orthosis checked if necessary.

Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy.

Take off the orthosis if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole, have not been reported to date. Correct fitting is assumed. Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the forearm and the hand.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin located remotely from the fitted aid.
4. Bottleneck syndromes (irritation due to compression of nerves in the arm).

Application instructions

Putting on the EpiPoint

Undo the band and place the EpiPoint strap on the forearm, ensuring the five-point pad rests directly on the muscle belly, i.e. in lateral position (outside)  in case of tennis elbow or in medial position (inside)  in case of golfer's elbow. The rigid part of the strap should always face outwards.

To fasten the strap, rest the forearm on a firm surface (table, etc.) and tighten the Velcro fastening  . Please avoid fastening the orthosis too tight! If the red warning section indicates excessive contact pressure, make sure to loosen the strap accordingly.

Product care and washing instructions

Please refer to the care instructions on the sewn-in label of your EpiPoint.

EpiPoint can be washed separately by hand at 30 °C using a mild detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaints, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications (elbow). See purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the EpiPoint have not been observed. Warranty is excluded if:

- The product was not used according to the indication

- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Thermoplastic Polyurethane (TPU), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Polyurethane (PUR), Polyoxymethylene (POM), Cotton (CO), Polyester (PES), Spring Steel Strip C75 / coated), Elastodiène (ED), Spandex (EL)

MD – Medical device
UDI – DataMatrix Unique Device Identifier

Version: 2022-11

1Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso
 2A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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LLC
Bauerfeind USA, Inc.