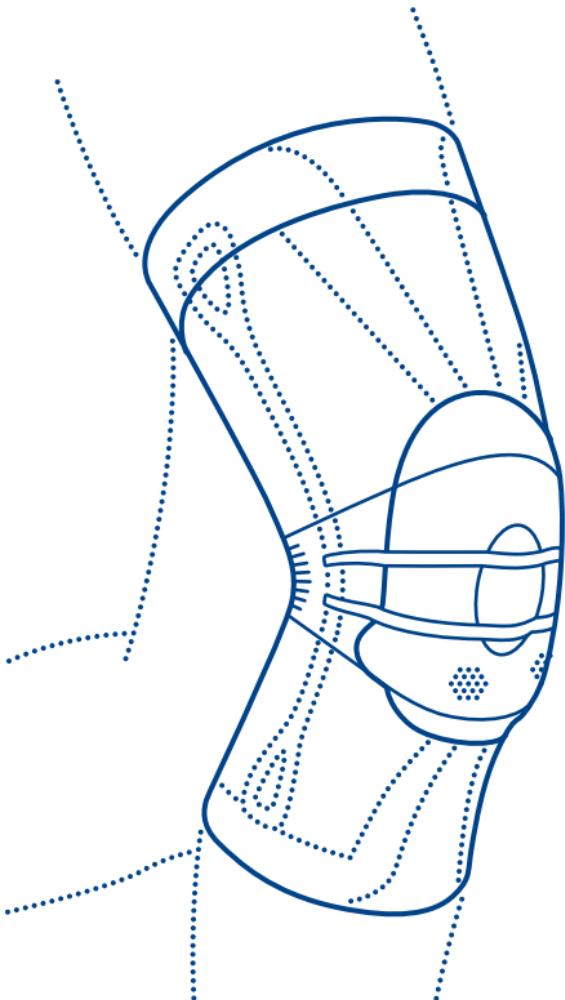
# GenuTrain®

KNIEBANDAGE

KNEE SUPPORT

GENOUILLÈRE

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Dear customer,  
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

#### Intended purpose

The GenuTrain is a medical device.  
It is a support for the relief and stabilization of the knee joint.

#### Indications

- Tendinopathy, tendomyopathy, ligament insertion degeneration, meniscopathy
- Chronic, post-traumatic or post-operative irritation
- Recurring joint effusion
- Gonarthrosis (joint wear)
- Osteoarthritis (joint inflammation)
- (Relapse) prevention
- Feeling of instability

#### Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the GenuTrain, only use it according to your indication and any additional instructions given by a medical specialist\*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

#### Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness, and excessive heat build-up
- Venous insufficiency, particularly with varicose veins
- Impaired sensation in the leg due to neurological abnormalities
- Impaired blood flow in the leg (e.g. with diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

#### Application instructions

##### Putting on the GenuTrain

- Grip the donning aids at the top ends of the stays on both sides and pull the support up over the knee.
- Position the Omega+ Pad (ring-shaped pad) in such a way that it encloses the kneecap without exerting pressure. Image ① ② ③

\* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**BAUERFEIND AG**