**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

LumboLoc Forte is a medical product. It is an orthosis<sup>1</sup> for increased stabilization and relief of the lumbar spine with a variable strap system.

## Indications

- Non-specific lower back pain (lumbago)
- Weak back / abdominal muscles (muscular insufficiency)
- Slight vertebral displacement (spondylolisthesis)
- Ligament / tendon / muscle irritation (ligamentotendomyopathies)

## Risks of using this product

- LumboLoc Forte is a prescribed product that should be worn under a physician's guidance. LumboLoc Forte should only be worn in accordance with the instructions provided and for the specified areas of application.
- The product should be worn over a T-shirt or an undershirt.
- LumboLoc Forte must be individually adapted to the patient by a trained specialist<sup>2</sup>. This is the only way to guarantee the full effect and optimal wearing comfort of the orthosis.
- Incorrect use of the support will terminate any product liability.
- No modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby terminating any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- Please consult your medical provider before using LumboLoc Forte in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- If you have any complaints, please contact your medical retailer only. We would point out that only clean goods can be dealt with.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.
- Loosening the strap / fastening can lead to relief from symptoms. CAUTION: Loosening the strap / fastening can limit the functional capacity of the orthosis. Please have the fit of the orthosis checked by a specialist without delay.

## Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

 Contains natural rubber latex (Elastodien) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

## Contra-indications

- Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and worn after consultation with your doctor:
  - Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
  - Impaired sensation.
  - Significant restriction of cardiopulmonary performance (risk of raised blood pressure if strenuous physical exertion is undertaken while wearing a therapeutic device).

## Application instructions

### Fitting and removing LumboLoc Forte

#### Fitting:

- Place the orthosis around the trunk so that the stays are positioned across the back, as shown in Fig. 1 (sewn-in labels at the top). The lumbar pad should be located mainly over the transition between the sacrum and the lumbar spine, and the sacral pad mainly below this point.

- Place the fingers in the specially provided finger pockets on the two broad, light-grey fastener halves and pull forward uniformly.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the velcro fastening (Fig. 2).
- When securing the velcro fastening, remove the fingers from the left finger pocket first, followed by the right (Fig. 3).
- Take the two lower LPT (Low Power Tension) straps in both hands and pull forward at the same time to produce the desired level of force (Fig. 4). Next, secure both LPT straps against the large velcro fastening (Fig. 5).
- **Correct fitting should be practiced with the patient.**

#### Remove:

- Unfasten the LPT tension straps and then refasten without any tension.
- Open the large, light-grey fastener.
- Remove the orthosis and refasten the light-grey fastener.
- For easier handling of LumboLoc Forte, always fasten any opened fasteners directly on the product.

## Cleaning instructions

The sewn-in label of the LumboLocForte, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the inside of the knitted fabric.

- We recommend washing LumboLoc Forte separately by hand with a mild detergent at 30 °C (86 °F). To avoid damage and to preserve their long-term adhesive strength, we recommend closing the velcro fastenings before washing.
- The knitted fabric and strap should be separated from each other for cleaning purposes.
- Remove the pad for cleaning. The pad can be cleaned with a damp sponge.

## Part of the body this product is used for

In accordance with the indications (lumbar spine). See intended use.

## Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the LumboLoc Forte have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

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## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyamide (PA), Polyurethane (PUR), Elastodiène (ED), Viscose (CV), Steel, Polyoxyéthylène (POM), Spring steel wire grade A, galvanized, Ethylene-vinyl acetate copolymer (EVAC), Polyester (PES)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2022-08


## Notes for the orthopedic technician

The height location of the fitted function strap can be adjusted by means of the velcro fastening.


The corset stays should be adjusted as necessary.

## Attaching the pad

- Two types of pads can be used depending on the indication: a lumbar or sacral pad.
- The pads must be velcro-fastened to the specially provided fleece belt on the inside of the support. For the lumbar pad (Fig. A), the wide end of the pad should always be at the top. The reverse applies to the sacral pad (Fig. B).
- The patient's anatomy should be taken into account when positioning the pad. Depending on the individual situation, the pad can be shifted up or down to achieve the correct fit.

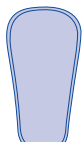**Fig. A**  
lumbar pad

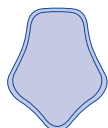**Fig. B**  
sacral pad

<sup>1</sup> Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

<sup>2</sup> A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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