
LumboTrain® straight | waisted

RÜCKENBANDAGE
BACK SUPPORT
ORTHÈSE LOMBAIRE
TUTORE PER LA SCHIENA

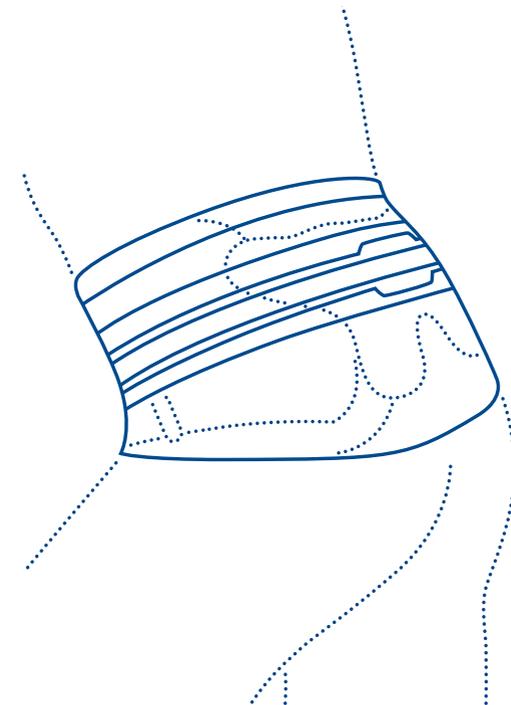86°F

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Dear patient,
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

LumboTrain straight / LumboTrain waisted are medical devices. The supports provide support for the back muscles for discomfort in the lumbar spine region.

Indications

- General pain in the lumbar spine region (acute and chronic)
- Degenerative conditions of the lumbar spine (e.g. osteochondrosis, osteoarthritis, lumbar deformities, spondylolysis without vertebral displacement)
- Status post prolapse (non-surgical / post-operative)
- Status post discectomy

Risks of using this product

- The benefits of this product are felt particularly during physical activity.
- Please remove the support during longer periods of rest.
 - Once you have been prescribed the LumboTrain straight / LumboTrain waisted, only use it according to your indication and any additional instructions given by a specialist*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
 - Avoid contact with ointments, lotions, or substances containing grease or acids.
 - Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
 - If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
 - If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Where to apply the product

See Indications and Intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is maintenance-free.

Assembly and fitting instructions

The product is supplied ready for use.

Technical specifications / parameters

- Your product comprises:
- Support with abdominal fastening
 - Dotted pad.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be

cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the LumboTrain straight / LumboTrain waisted have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily.

Disclaimer

For use of the LumboTrain straight / LumboTrain waisted, the following applies: Do not diagnose yourself or decide on which medication to take unless you are a specialist*. Before first using our product, please seek the advice of this specialist*, as this is the only way to assess the effect of our LumboTrain straight / LumboTrain waisted on your body and determine the potential risks of using this product due to

Contraindications

- No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness, and excessive heat build-up
 - Impaired sensation
 - Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is applied during intense physical exertion).

Application instructions

Applying the LumboTrain straight / LumboTrain waisted

- Fasten the pad to the center of the Velcro surface of the support so that the opening for the silicone dots runs vertically down the center. 
- Place your fingers in the hand straps and the support around the waist.  The pad must sit in the middle of the back.
- Pull the two halves of the fastening forward evenly and guide them one over the other: press the left side against the stomach and attach the right side. When fastening the support, remove your fingers from the left hand strap.  

Cleaning instructions

Please use mild detergent and a lingerie bag. Remove the pad and wash it by hand. Close all the Velcro fastenings. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

your personal constitution. Follow the advice of the specialist* as well as all instructions in this brochure or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the local health authorities. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content LumboTrain straight

Polyamide (PA), Thermoplastic elastomer (SEBS Compound), Polypropylene (PP), Thermoplastic polyurethane (TPU), Elastane (EL), Polyurethane (PUR), Cotton (CO), Polyester (PES), Polyethylene (PE), Polydimethylsiloxane polyurea block copolymer (PDHC)

LumboTrain waisted

Polyamide (PA), Thermoplastic elastomer (SEBS Compound), Polypropylene (PP), Thermoplastic polyurethane (TPU), Polyurethane (PUR), Elastane (EL), Cotton (CO), Polyester (PES), Polydimethylsiloxane polyurea block copolymer (PDHC)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

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* A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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Bauerfeind Ibérica, S.A.
Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.