AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Styren-Ethylen/Butylen-Styren-Blockcopolymer (TPS-SEBS), Polyamid (PA), Ethylen-Vinylacetat (EVA), Polyoxymethylen (POM), Polypropylen (PP), Polyester (PES), Polyurethan (PUR)

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en english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MalleoLoc is a medical device. It is an orthosis for increased stabilization of the upper and lower ankle.

Indications

- Early functional / non-surgical treatment for injuries to the capsular ligament complex and the bifurcate ligament, e.g. after sprains and ruptured ligaments
- Post-operative protection after ligamental suturing / reconstruction
- Chronic ligament insufficiency
- Relapse prevention

⚠ Application risks

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

- The use and duration of use must be discussed with the physician who prescribed the MalleoLoc. He will also inform you whether the MalleoLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the MalleoLoc as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the MalleoLoc – e.g. an increase in discomfort – stop using it and contact a physician immediately.
- Adjust your behavior to the situation associated with your specific handicap / treatment need and any resulting restriction of movement, and avoid anything that could delay recovery where possible. Proceed with particular caution.
- If your perceptive faculties are restricted, if you take painkillers or blood-thinning

medication or suffer from dementia, our products must only be worn when prescribed by a physician and under constant supervision of trained specialists?

- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- Before using the product, the specialist? or your physician must determine whether thrombosis prevention is necessary.
- If you notice any negative changes or an increase in symptoms while using the product, stop any further use and contact a physician.
- Furthermore, proceed with extreme caution, as your ability to stand and walk (for hand and arm products your ability to grip) can also be heavily impacted.
- For products that affect the locomotor apparatus, check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the part of the body that is receiving treatment. If the ground is smooth, there is an acute risk of slipping and thus injury!

Contraindications

- No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
 - Impaired sensation and circulatory disorder of the legs or feet, e.g. in diabetes mellitus
 - Impaired lymph drainage. including soft tissue swelling of uncertain origin located some distance away from the orthosis.

Application instructions

Putting on / taking off the MalleoLoc

Sit down to put on the MalleoLoc.

- 1 Open the two velcro straps.
- 2 Push the loop of the lower strap down and out of the holder, to loosen the strap.
- 3 Turn the MalleoLoc so that the straps point inwards. The projection on the lower end of the shell points forwards.
- 4 Now place the foot centrally in the shell of the MalleoLoc. The inner part of the stay will be located behind the centre of the ankle **4a** and the outer part will be in front of the centre of the ankle **4b**.
- 5 Close the upper strap by placing it around the ankle joint from the back. When applying the strap, make sure that it sits without folds. The lower strap is now placed around the foot and lower leg in the shape of a figure of eight. To do this, proceed as follows:
- 6 Place the strap diagonally over the back of the foot, towards the back, around the lower leg.
- 7 Now guide the strap diagonally forwards once more, to the outer side of the foot, and
- 8 push the loop from the bottom to the top, into the holder.
- 9 Fasten the end of the strap upwards onto the part of the strap lying underneath.

Cleaning instructions

Note: Never expose the MalleoLoc to direct heat (e.g. heaters, sunlight, in your car etc.!) This may damage the material. This can impair the effectiveness of the MalleoLoc. For cleaning, remove the straps from the MalleoLoc.

To avoid damage to the product, wipe the shell of the MalleoLoc with a mild detergent only.

Fasten the Velcro fastening. Wash the straps by hand.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Assembly and fitting instructions

The MalleoLoc is supplied ready for use.

Assembling the MalleoLoc after cleaning:

Fitting the loop:

1 Hold the loop with the tip pointing downwards and the Bauerfeind logo "B" forwards between thumb and forefinger. Now guide the strap with the light gray broad side through the loop from the front.

Fitting the straps:

Place the long strap that you have just fitted with the loop in front of you, with the dark gray side upwards. 1 Guide the strap with the narrow side from the outside inwards, through the lower (closed) part of the MalleoLoc shell. 2 Now guide the short strap with the dark gray side pointing upwards from the inside outwards, through the open part of the MalleoLoc shell.

Technical specifications / parameters

The MalleoLoc is an orthotic support for the ankle with two straps attached to it. The straps are removable. In addition, a loop for fastening is fitted to the lower (longer) strap. The two straps and the loop are available as spare parts.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist* were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Polyamide (PA), Ethylene Vinyl Acetate (EVA), Polyoxymethylene (POM), Polypropylene (PP), Polyester (PES), Polyurethane (PUR)

- MD – Medical Device
- UDI – DataMatrix Unique Device Identifier UDI

Version: 2023-06

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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