AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ManuLoc Rhizo / ManuLoc Rhizo long are medical devices. They are orthoses³ for immobilization of the wrist (long version: and parts of the forearm) with thumb brace.

If the product is not used properly, all warranty rights will be rendered null and void.

Indications / Areas of Application

ManuLoc Rhizo

- Acute, post-traumatic, and post-operative irritation in the wrist and thumb ray
- Pre-operative care
- Tenosynovitis at the radial side (moderate)
- Osteoarthritis of the carpometacarpal joint of the thumb (severe)
- Carpal tunnel syndrome (mild)
- Sprains / contusions
- Ligament injuries in the finger / hand area

ManuLoc Rhizo long

- Post-traumatic or post-operative irritation, e.g. following treatment with a plaster cast in the case of scaphoid fractures
- Pre-operative care
- Tenosynovitis at the radial side (moderate)
- Osteoarthritis of the carpometacarpal joint of the thumb (severe)
- Sprains / contusions
- Carpal tunnel syndrome (moderate)
- Collateral ligament injuries, e.g. skier's thumb

Risks of using this product

Caution¹

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

- The ManuLoc Rhizo / ManuLoc Rhizo long must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (intended use, parts of the body) listed.
- To ensure the optimum fit of the ManuLoc Rhizo / ManuLoc Rhizo long, the size of ManuLoc Rhizo / ManuLoc Rhizo long required must be determined by a trained specialist².
- The sewn-in label of the ManuLoc Rhizo / ManuLoc Rhizo long, which contains information on the size, manufacturer, washing instructions and CE mark, is located in the insertion opening for the wide stay located on the inside of the hand (the volar stay). Only a trained specialist² may carry out the (initial) fitting of the orthosis and provide instruction.
- Use in combination with other products, e.g. as part of compression treatment, must be discussed beforehand with your physician.
- Should you experience increased discomfort or notice any unusual changes, please contact your physician without delay.
- No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed. Any medical aids – such as supports and orthoses³ – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict the underlying blood vessels or nerves. If this happens, loosen the straps of the orthosis a little and, if necessary, check the size of your orthosis.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.

- Impaired sensation and circulatory disorders of the arm / hand, e.g. in the case of diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the ManuLoc Rhizo / ManuLoc Rhizo long

- 1 Start by loosening the three Velcro straps on the lower arm splint (or the four Velcro straps on the long version).
- 2 Place the ManuLoc Rhizo / ManuLoc Rhizo long on the correct hand. The straps are thus located on the top of the lower arm and on the back of the hand.
- 3 Fasten the straps on the lower arm splint in the following sequence: Lower arm straps 1, 2 and the strap over the back of the hand (or lower arm straps 1, 2, 3 and the strap over the back of the hand on the long version).
- 4 Close the Velcro closure on the thumb attachment.

Taking off the ManuLoc Rhizo / ManuLoc Rhizo long

Open the lower arm straps, the strap over the back of the hand and the thumb closure. Take off the ManuLoc Rhizo / ManuLoc Rhizo long.

Cleaning instructions

Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
- Please only wash the orthosis¹ by hand at 30 °C using a mild detergent. All components can stay on the orthosis¹ during washing. Leave the orthosis¹ to air dry.
- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the sewn-in label on the upper edge of your product.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your retailer. Only clean goods are handled.

Where to apply the product

see Indications and Intended purpose

Maintenance instructions

If handled and cared for correctly, the product is maintenance-free.

Assembly and fitting instructions

The ManuLoc Rhizo / ManuLoc Rhizo long is supplied in an anatomically contoured state.

Technical specifications / parameters

The ManuLoc Rhizo / ManuLoc Rhizo long is a stabilizing orthosis for the wrist. It consists of a long lower arm splint. The straps on the lower arm splint are detachable. The straps and thumb support are available as spare parts on request.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc Rhizo / ManuLoc Rhizo long have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Aluminium (AlMg3), Polyurethane (PUR), Polyether, Polyoxymethylene (POM), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2022-04

Specialists²

General instructions

- Check the optimal and individual fit of the ManuLoc Rhizo / ManuLoc Rhizo long at the first fitting, and adjust if necessary.
- Practice the correct procedure for putting on the support with the patient.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, so that product liability is excluded.
- The ManuLoc Rhizo / ManuLoc Rhizo long is intended for the treatment of one patient only. The aluminum stays are anatomically contoured, but they can still be individually adjusted to the patient.

Shaping the stays:

- 1 Carefully remove the stays from the insertion pockets and shape them to the anatomy of the patient. Reinsert the three stays into the pockets provided.

Adjust the thumb attachment to the length and position of the patient's thumb:

- 1 Loosen the lower Velcro closure between the hand orthosis and thumb attachment. Adjust to the patient's thumb length by moving the attachment within the channels provided. The thumb abduction angle is directly set by shaping the integrated aluminum stays and by the Velcro fastening on the thumb. Press the lower Velcro fastening down firmly after making adjustments.

¹ Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product)

² A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

³ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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