AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

Bauerfeind Benelux B.V.
Waaarderveldweg 1
2031 BK Haarlem

THE NETHERLANDS

P +31 (0) 23 531-9427
F +31 (0) 23 532-1970
E info@bauerfeind.nl

BELGIUM

P +32 (0) 2 527-4060
F +32 (0) 2 792-5345
E info@bauerfeind.be

BOSNIA AND HERZEGOVINA

Bauerfeind d.o.o.
Meše Selmovića 19
71000 Sarajevo
P +387 (0) 33 710-100
F +387 (0) 33 619-422
E info@bauerfeind.ba

CROATIA

Bauerfeind d.o.o.
Goleška 20
10020 Zagreb
P +385 (0) 1 6542-855
F +385 (0) 1 6542-860
E info@bauerfeind.hr

FRANCE

Bauerfeind France S.A.R.L.
B.P. 59258
95957 Roissy CDG Cedex
P +33 (0) 1 4863-2896
F +33 (0) 1 4863-2963
E info@bauerfeind.fr

ITALY

Bauerfeind Italia Srl
Via Cornaggia 58
20092 Cinisello Balsamo (MI)
P +39 02 8977 6310
F +39 02 8977 5900
E info@bauerfeind.it

NORDIC

Bauerfeind Nordic AB
Storgatan 14
114 55 Stockholm
P +46 (0) 774 100 020
E info@bauerfeind.se

REPUBLIC OF NORTH MACEDONIA

Bauerfeind Dooel Skopje
50 Divizija 24 a
1000 Skopje
P +389 (0) 2 3179-002
F +389 (0) 2 3179-004
E info@bauerfeind.mk

SERBIA

Bauerfeind d.o.o.
102 Omladinskih brigada
11070 Novi Beograd
P +381 (0) 11 2287-050
F +381 (0) 11 2287-052
E info@bauerfeind.rs

SINGAPORE

Bauerfeind Singapore Pte Ltd.
Blk 41 Cambridge Road #01-21
Singapore 210041
P +65 6396-3497
F +65 6295-5062
E info@bauerfeind.com.sg

SLOVENIA

Bauerfeind d.o.o.
Dolenjska cesta 242 b
1000 Ljubljana
P +386 (0) 1 4272-941
F +386 (0) 1 4272-951
E info@bauerfeind.si

SPAIN

Bauerfeind Ibérica, S.A.
C / San Vicente Mártir,
nº 71 – 4º - 7ª
46007 Valencia
P +34 96 385-6633
F +34 96 385-6699
E info@bauerfeind.es

SWITZERLAND

Bauerfeind AG
Vorderi Böde 5
5452 Oberrohrdorf
P +41 (0) 56 485-8242
F +41 (0) 56 485-8259
E info@bauerfeind.ch

UNITED ARAB EMIRATES

Bauerfeind Middle East FZ
LLC
Dubai Healthcare City
Building 40, P.O. Box 505116
Dubai
P +971 4 4335-684
F +971 4 4370-344
E info@bauerfeind.ae

UNITED KINGDOM

Bauerfeind UK
85 Tottenham Court Road
London
W1T 4TQ
P +44 (0) 121 446-5353
F +44 (0) 121 446-5454
E info@bauerfeind.co.uk

USA

Bauerfeind USA, Inc.
75 14th St NE
Suite 2350
Atlanta, GA 30309
P +1 800 423-3405
P +1 404 201-7800
F +1 404 201-7839
E info@bauerfeindusa.com








 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM

OmoTrain®

Schulterbandage
Shoulder support
Épaulière
Tutore per la spalla
Vendaje para el hombro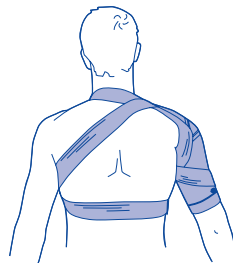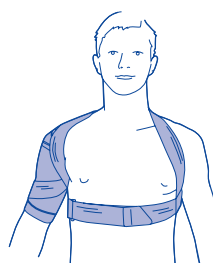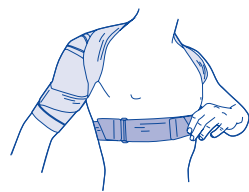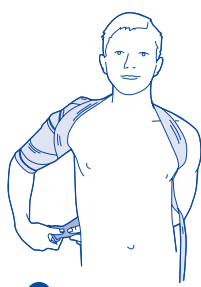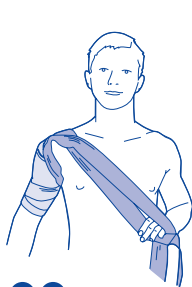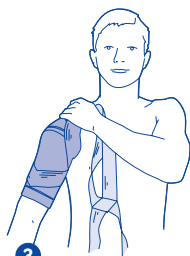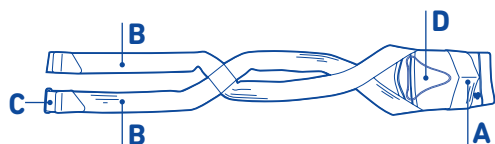

de deutsch	3	sr srpski	23
en english	4	sl slovenski	25
fr français	5	ro romanian	26
nl niederlands	7	el ελληνικά	27
it italiano	8	tr türkçe	28
es español	9	ru русский	30
pt português	11	et eesti	31
sv svenska	12	lv latviešu	32
no norsk	13	lt lietuvių	33
fi suomi	14	uk українська	35
da dansk	16	he עברית	37
pl polski	17	ja 日本語	38
cs česky	18	ko 한국어	39
sk slovensky	20	ar عربي	41
hu magyar	21	zh 中文	42
hr hrvatski	22		

de deutsch

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Polyamid (PA), Styren-Ethylen / Butylen-Styren-Blockcopolymer (TPS-SEBS), Elastan (EL), Elastodien (ED), Viskose (CV), Polyoxymethylen (POM), Polyurethan (PUR), Polyester (PES), Baumwolle (CO)

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en english

Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Purpose

OmoTrain is a medical device. It is a support for the relief and stabilization of the shoulder joint.

Indications

- Chronic, post-traumatic or post-operative irritation
- Prevention / relapse prevention
- Osteoarthritis (joint wear)

⚠ Risks of using this product Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the OmoTrain, only use it according to your indication and any additional instructions given by a medical specialist¹. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.
- To ensure the optimum fit of the OmoTrain, the size of OmoTrain required must be determined by a trained specialist¹.

⚠ LATEX Contains natural rubber latex (elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions such aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and movement disorders of the arms (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support
- Vein occlusion in the arm (arm vein thrombosis)

Application instructions

Putting on the OmoTrain
A – Knitted section of the support,
B – Velcro straps, C – Eyelet,
D – Massage pad (Delta pad)
1 Fasten one the OmoTrain straps and pull it out through the eyelet. To avoid damaging your clothes or the OmoTrain, fasten the ends of the strap loosely onto the strap underneath it.
2 Secure the knitted section onto the fastened straps. The opening should be pointing toward you.
3 Then guide the affected arm through the knitted section of the support until the shoulder is well covered. The massage pad (Delta pad) should be resting on the bony protuberance at front and back of the shoulder. If this is not the case, adjust the position of the support or the massage pad.
4 Arrange both straps in a flat (without twisting), diagonal position across the stomach.
5 Take hold of them by inserting your hand through the loop from the inside.
6 Pull the loop behind your head and slip your arm through it.
7 Guide the rear strap across your back toward the affected arm and secure it in place.
8 Then guide the front strap around your abdomen toward the affected side. Pull the strap that you unfastened to begin with back through the eyelet and fasten it.
9 Then tighten the abdominal strap until the required degree of stabilization is achieved. Fasten the end of the strap.

Taking off the OmoTrain

Undo both straps. To avoid damaging your clothes or the OmoTrain, fasten the ends of the straps loosely. Slip your arm out of the sling first, and then slip it out of the knitted section of the support.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Refasten the massage pad after washing. To do this, lay the OmoTrain with the straps facing upward on the long side of the knitted fabric. Fasten the massage pad onto the knitted section with the long tapered end pointing toward you.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OmoTrain have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Spandex (EL), Elastodiene (ED), Viskose (CV), Polyoxymethylene (POM), Polyurethane (PUR), Polyester (PES), Cotton (CO)

MD – Medical Device

UDI – DataMatrix Unique Device Identifier UDI

Version: 2023-03

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports which are relevant to you.

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