30°C
86°F


**BAUERFEIND AG**

Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com

BAUERFEIND.COM**ValguLoc®**

Hallux-valgus-Orthese
Hallux valgus orthosis
Orthèse hallux valgus
Tutore per alluce valgo
Ortesis hallux valgus

MADE
IN
GERMANY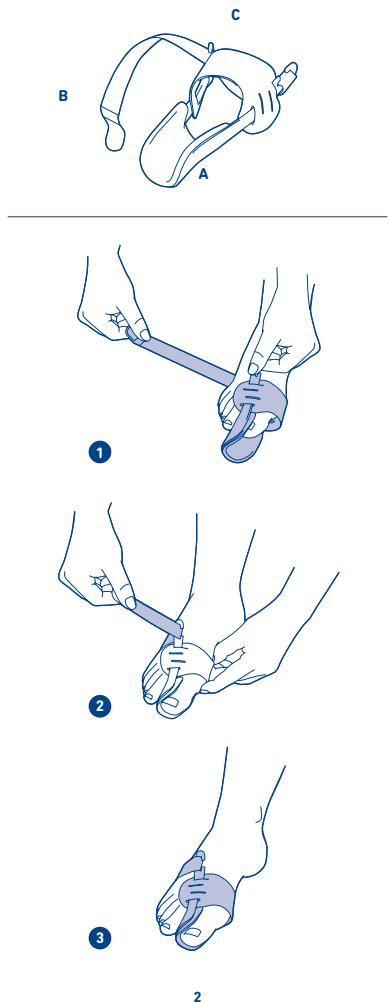

| | | |
|----|------------|----|
| de | deutsch | 3 |
| en | english | 6 |
| fr | français | 9 |
| nl | nederlands | 12 |
| it | italiano | 15 |
| es | español | 18 |
| pt | português | 21 |
| sv | svenska | 24 |
| no | norsk | 27 |
| fi | suomi | 30 |
| da | dansk | 33 |
| pl | polski | 35 |
| cs | česky | 39 |
| sk | slovensky | 41 |
| hu | magyar | 44 |
| hr | hrvatski | 47 |
| sr | srpski | 50 |
| sl | slovenski | 53 |
| ro | romanian | 56 |
| el | ελληνικά | 59 |
| tr | türkçe | 62 |
| ru | русский | 65 |
| et | eesti | 68 |
| lv | latviešu | 70 |
| lt | lietuvių | 73 |
| uk | українська | 76 |
| he | עברית | 80 |
| ja | 日本語 | 82 |
| ko | 한국어 | 85 |
| ar | عربى | 88 |
| zh | 中文 | 91 |

de deutsch

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 en english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended use

ValguLoc is a medical device. It is an orthosis¹ for the correction of the big toe at rest.

⚠ The ValguLoc must be individually adjusted for the patient by a trained specialist³. This is the only way to guarantee its full effect and optimal wearing comfort.

Indications

• Hallux valgus (bending of the big toe toward the little toe) – conservative and post-operative treatment.

Risks of using this product

⚠ Caution²

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

The ValguLoc must be worn in accordance with these instructions and in the specified areas (where to fit the orthosis).

The ValguLoc is a night splint. It is **not** suitable for use while walking.

To guarantee the best possible fit of the ValguLoc, it is essential to check whether the product is adequately shaped, or whether it needs to be reshaped by a specialist³ to fit the body. Only a trained specialist may carry out the (initial) fitting of the orthosis and provide instruction. Consult your physician before combining with other products e.g. as part of compression therapy (compression stockings).

Should you notice increased symptoms or any unusual changes, stop using the product and please contact your physician without delay.

Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby excluding any product liability.

No side effects that affect the entire body have been reported to date. This assumes the orthosis is used / fitted correctly.

Do not fit the ValguLoc too tightly, as this could lead to local pressure points. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps on the ValguLoc and have the size of your ValguLoc checked if necessary. If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify.

In particular, the ability to operate vehicles, other means

of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the part of the body being treated, particularly if inflammation occurs. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
- Impaired sensation and circulatory disorders in the leg / foot, e.g. diabetes mellitus
- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn.
- Gout (arthritis urica)

Application instructions

Putting on / applying the ValguLoc

A – Plastic brace, B – Strap with Velcro fastening, C – Ball tab

1 Open the Velcro fastening on the ValguLoc strap. Push the plastic brace between the big toe and the second toe until the big toe is lying in the hollow of the ValguLoc. **2** Now feed the strap through the opening in the brace and secure it in place tightly with the Velcro fastening. **3** In doing so, make sure that the position of the big toes looks correct without causing pain.

Taking off / removing the ValguLoc[®]

Open the Velcro fastening on the strap and move the foot out of the ValguLoc.

Cleaning instructions

Note: Never expose the ValguLoc to direct heat (e.g. heaters, sunlight, leaving in your car, etc.) This may damage the material, which can impair the effectiveness of the ValguLoc.

Close all Velcro straps before cleaning to avoid damage. Clean the ValguLoc separately by hand with a mild detergent.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Where to apply the orthosis

In accordance with the indications (big toe). Please see intended use.

Maintenance instructions

If handled and cared for correctly, the product requires virtually no maintenance.

Assembly and fitting instructions

The ValguLoc is delivered in an anatomically (ergonomically) preshaped state. If necessary, the ValguLoc can be shaped further. The orthoses come in two versions for the right and left big toe.

Technical details / parameters

The ValguLoc is an orthosis for the big toe.

Replacement straps are available.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on

how to handle and care for the ValguLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA),
Ethylene Vinyl Acetate (EVA),
Acrylate, Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2023-02

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

² Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

³ A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

Specialists³

General instructions

The first time it is used, check that the ValguLoc fits the individual patient optimally. Make any necessary corrections.

Practice the correct procedure for putting on the orthosis with the patient.

No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby excluding any product liability.

The ValguLoc is intended for treating one patient only.

The ValguLoc is anatomically (ergonomically) preshaped, but may still need to be individually adjusted to fit the patient's body. To do this, apply a little force to form the orthosis into the anatomically correct shape.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.