AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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(en) english

Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoPed is a medical device for the feet. It is a viscoelastic foot orthosis for pressure relief in the forefoot, midfoot and heel area and for gentle correction of static or dynamic misalignment of the skeleton of the foot.

Indications

- Mild skew foot / flat foot deformities
- Splay foot
- Forefoot and toe deformities (around the sole of the foot)
- Pressure pain around the sole of the foot
- Local pressure peaks
- Joint pain

Risks of using this product

- ViscoPed is a prescribed product that should be applied under a physician's guidance. ViscoPed should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- No liability is accepted for improper use of the product.
- Should you notice increased symptoms or any unusual changes, please stop using the product and please contact your physician without delay.
- Never expose the ViscoPed to direct heat (e.g. heater)! This may cause damage to the material, which can impair the effectiveness of the ViscoPed.

- Side effects involving the body as a whole have not been reported to date.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.

The inlay is not suitable for diabetics who have diabetic foot syndrome.

Fitting ViscoPed

 ViscoPed must be fitted for the first use by a trained specialist¹, otherwise we cannot accept any liability for the product.

1. Place the ViscoPed insoles into the shoes as marked (R = right, L = left) so that they are flush with the heel area of the shoes.
2. If the ViscoPed foot orthoses are too long, the marked toe area can be shortened using shears.
3. Place the fitting aid on the insole with the fluffy side downwards and slip foot into the shoe. Remove fitting aid.
4. Unless otherwise prescribed by the physician treating you, insert both the left and right version of the foot orthosis into your shoes.

Product care

We recommend washing ViscoPed handwarm with a mild detergent.

Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoPed have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical device
 – DataMatrix Unique Device Identifier

Version: 2023-06

¹ A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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