Knee Support
Kniebandage

B101

General guidelines

Take the time to thoroughly read the following instruction and utilize the product accordingly.

Ensure you retain this manual and pass it along when transferring the product.

Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

Warnings

To prevent the risk of suffocation, keep all parts and packaging materials (such as films, plastic bags, and foam) away from babies and children.

While assembling, keep small parts out of the reach of children. Swallowing or inhaling these parts can be fatal.

It is strictly forbidden to use the product for anything other than its intended use. Children must be supervised by adults when using the product.

Alterations or modifications to the product without authorization can compromise safety.

Do not use the product if you have a disability or are taking medication that may interfere with safe use of this product.

Clean the product with a mild soap or household cleaner. Do not use organic solvents (such as acetone, lacquer thinner, lighter fluid, dry cleaning fluid, petrol or turpentine) that may adversely affect the product.

Notes

If you wear the brace at night, we recommend that you take it off for a few hours afterwards, so that you can still put pressure on the knee. If you still need support during the day, you can put the brace back on later.

Check the parts carefully for damage before use. Incorrect use or damaged parts pose a risk of injury.

Note: Place the product on or around the part of the body for which it is intended. Under no circumstances use the product for any part of the body other than that for which it is intended.

Do not use this product 24/7 for more than a few weeks to avoid weakening of the knee.

Keep the product away from sharp objects.

This product is intended for use by one person.

This product is only suitable for hand washing at low temperatures (30 degrees). It is not suitable for the washing machine and dryer.

Use only authentic Dunimed parts if you need to replace any part of the product.

Store this product at room temperature in a dry, ventilated place to prevent mould growth.

Always ensure that the product is not so tight that it pinches off certain limbs.

The lifespan of a brace with frequent use is usually 1-2 years, depending on the brand and type. Regular use leads to wear and reduced support. Occasional use extends the lifespan. If you experience that the brace no longer provides adequate support, we recommend replacing the product.

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Podobrace B.V.