---

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| **GS005P**

## General guidelines

Take the time to thoroughly read the following instruction and utilize the product accordingly.

Ensure you retain this manual and pass it along when transferring the product.

Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

## Warnings

Keep all parts and packaging materials (such as foil, plastic bags and foam) away from babies and children to avoid suffocation hazards.

It is strictly forbidden to use the product for anything other than its intended use. Children must be supervised by adults when using the product.

Changes or modifications to the product without permission may compromise safety.

Do not use the product if you have a disability or are taking any medication that may interfere with the safe use of this product.

In case of pain, tingling, swelling, skin discolouration or other unusual symptoms during use, stop using the stockings and consult a doctor.

Do not use the compression stockings over open wounds or untreated skin conditions. This may impede the healing process or cause infections.

## Notes

Long-term use of compression stockings without medical advice may have unwanted effects. Consult a doctor before long-term use.

Take extra care when used by children and the elderly. They may be more sensitive to compression stockings.

Persons with certain medical conditions, such as severe peripheral neuropathy or arterial diseases, should be careful when using compression stockings.

Do not apply lotions, creams or ointments directly under the stockings unless recommended by a doctor, as this may damage the stockings or cause skin irritation.

Avoid using irons or drying with high heat, this may damage the elasticity of the stockings.

This product is only suitable for machine wash and hand wash at low temperatures (30 degrees). It is not suitable for tumble drying.

Do not pull or stretch the top edge of the stockings as this may reduce their effectiveness.

Make sure the stockings do not roll or fold while wearing, as this can cause pressure points.

Store this product at room temperature in a dry, ventilated place to prevent mould growth.

The lifespan of compression stockings with frequent use is usually 3-6 months, depending on the brand and type. Regular use leads to reduced elasticity and effectiveness. Occasional use extends the lifespan. If you experience that the stockings no longer provide adequate compression, we recommend replacing the product.

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