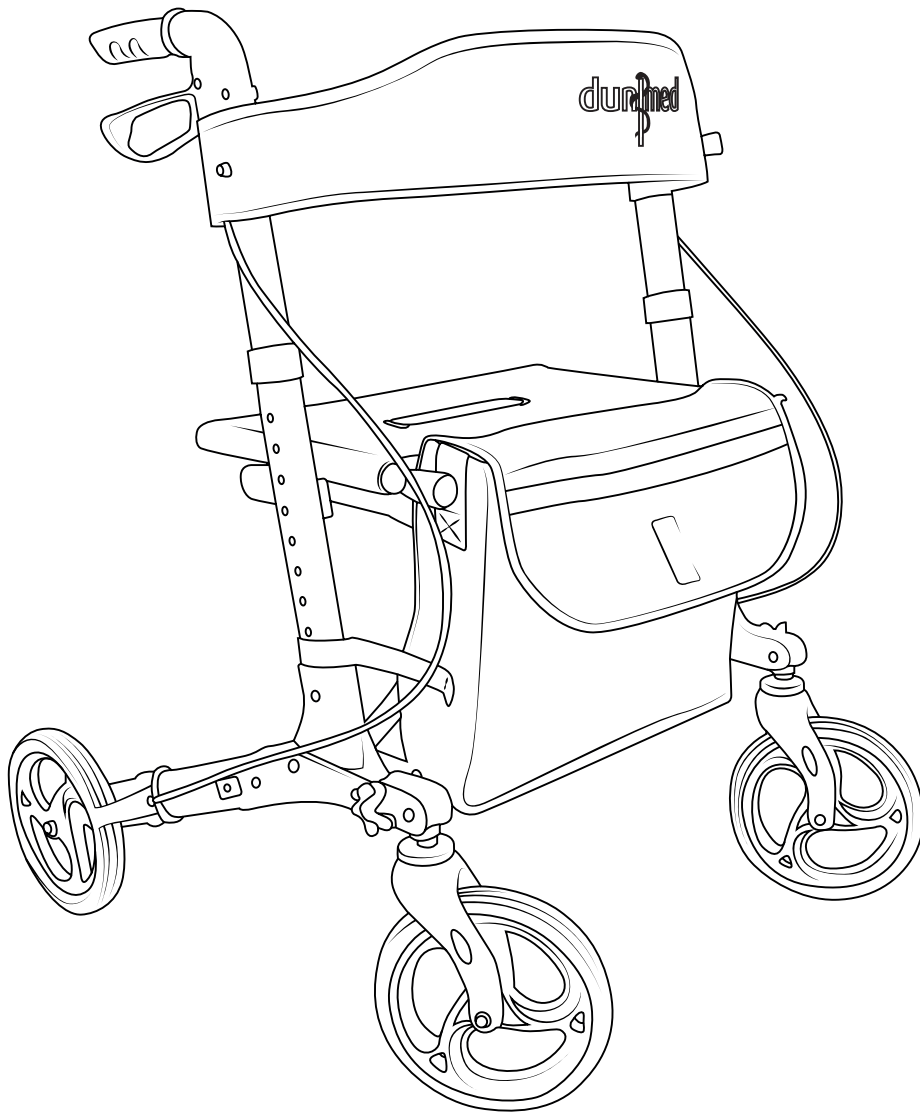
Rollators
Rollator

| **SC5025A**

General guidelines

Take the time to thoroughly read the following instruction and utilize the product accordingly.

Ensure you retain this manual and pass it along when transferring the product.

Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

Warnings

To prevent the risk of suffocation, keep all parts and packaging materials (such as films, plastic bags, and foam) away from babies and children.

Alterations or modifications to the product without authorization can compromise safety.

Do not use the rollator on stairs unless otherwise instructed by a professional.

The use of a corroded rollator may result in damage to the rollator and/or injuries to the user.

Do not use your rollator if you have any handicaps or disabilities or using medications, etc. that would impair or interfere with safe usage of this product.

Clean the grips with a mild soap detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

Notes

The maximum static load capacity is 136 kg (300 lbs), this must not be exceeded.

Prior to assembly, carefully inspect the components for any damage and strictly follow the provided instructions. Incorrect assembly or damaged components pose a risk of injury.

Always completely unfold the rollator before use. Make sure the rollator cannot fold inadvertently during use.

Under no circumstances should you use the product if any component is missing or defective.

Note: Please place the rollator on a level surface and ensure the base is in full contact with the ground. Engaging in dangerous actions such as playing or stooping on the rollator may cause imbalance.

Before using the seat, check that the rollator is on a flat, stable surface. It is essential for the brakes to be in the "parking brakes" position when using the seat.

Never stand on the rollator or use it as a ladder or climbing tool.

Avoid exposing the product to direct sunlight and too much water and keep it away from sharp objects.

Only use the rollator as instructed by your healthcare provider.

Only use authentic Dunimed parts if you need to replace any part of the rollator.

Store the rollator at room temperature, in a dry, ventilated place.

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