**INSTRUCTIONS FOR USE****454**

WRIST BRACE / ADJUSTABLE

**INTENDED USE**

Best for relieving pain from conditions such as carpal tunnel, arthritis, tendonitis and sprains.

**SIZE**

ONE SIZE FITS MOST

**WARNINGS & PRECAUTIONS**

This product is not a substitute for medical care. Always seek professional medical advice for the diagnosis and treatment of pain, injury or irritation. Neoprene supports are intended for use during short periods (several hours) of activity. Do not wear for more than 2-3 hours. Do not wear while sleeping. While being worn, especially during initial usage, regularly inspect the skin under the support. If a rash or other skin problems develop, discontinue use. If the problem persists, consult a physician. Do not apply over open wounds or if you have a susceptibility to dermatitis, an allergy to rubber or circulation problems

**DIRECTIONS**

1. Be sure you have the correct wrist – right or left
2. Place thumb through holes with rigid section positioned on palm-side
3. Pull top strap around and across palm-side and secure to backside above thumb
4. Wrap bottom strap across palm-side to fit and secure backside

**CONTENTS**

The product is provided nonsterile and is intended to be reusable on the same person multiple times. The product is manufactured out of 100% latex-free 3.2mm neoprene (CR). Exclusive of binding/cover fabric. Not made with Natural Rubber Latex.

**CLEANING**

Remove splint through a slit in neoprene. Hand wash in warm water with a mild detergent, rinse thoroughly and drip dry. **DO NOT MACHINE DRY.** Frequent washing is recommended to keep this support clean and free of body salts that may cause skin irritations.

**STORAGE**

When not in use, store in a cool, dry place.

**DISPOSAL**

Check product for excessive wear prior to each use. Discard the device if it shows signs of excessive wear and replace with a new product as necessary.

EN

FR

DE

IT

ES

PT

NL

BG

HR

CS

DA

ET

FI

EL

HU

LV

LT

PL

RO

SK

SL

SV

NO

## INSTRUCTIONS FOR USE

# 454

WRIST BRACE / ADJUSTABLE

### WHEN TO CONSULT A HEALTHCARE PROFESSIONAL

Consult a healthcare professional if experiencing any adverse reaction to the device including skin reaction or changes in condition.

### NOTICE TO USER AND/OR PATIENT

In the event of an adverse incident related to the device, remove the device and report the incident to the manufacturer.

### LABEL SYMBOLS

The symbols used on the product label or product literature are defined in the Symbols Legend here: [www.usb-eifu.eu/symbols](http://www.usb-eifu.eu/symbols)

EN

FR

DE

IT

ES

PT

NL

BG

HR

CS

DA

ET

FI

EL

HU

LV

LT

PL

RO

SK

SL

SV

NO

 United Sports Brands, Inc.  
11488 Slater Ave.  
Fountain Valley, CA 92708 USA  
For Info:  
[www.McDavidUSA.com](http://www.McDavidUSA.com)  
+1-800-233-6956

 United Sports Brands  
Janssen-Pharmaceuticaan  
Post Box 4  
2440 Geel, Belgium  
For Info: [www.McDavid.eu](http://www.McDavid.eu)

 CH Imported by:  
United Sports Brands  
Switzerland GmbH  
Westbahnhofstrasse 1  
CH-4500 Solothurn, Switzerland  
+32 (0) 14 85 47 85

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eIFU Ref: PN ART EU – MDXXX-IFU Rev 01 © 2024
QNET B.V.
QNET CH-REP GmbH,
QNET LTD.