INSTRUCTIONS FOR USE


463

SHOULDER WRAP

INTENDED USE

Best for helping to relieve pain and promote healing for non-specific shoulder injuries, arthritis and bursitis.

SIZE

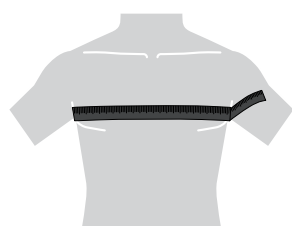SIZE:

Inch
Cm

S

30-36
76-91

M

36-42
91-107

L

42-50
107-127

WARNINGS & PRECAUTIONS

This product is not a substitute for medical care. Always seek professional medical advice for the diagnosis and treatment of pain, injury or irritation. Neoprene supports are intended for use during short periods (several hours) of activity. Do not wear for more than 2-3 hours. Do not wear while sleeping. While being worn, especially during initial usage, regularly inspect the skin under the support. If a rash or other skin problems develop, discontinue use. If the problem persists, consult a physician. Do not apply over open wounds or if you have a susceptibility to dermatitis, an allergy to rubber or circulation problems

DIRECTIONS

1. Check for 2 straps and 1 neoprene cuff
2. Place one end of long strap to backside on upper part of neoprene cuff
3. Slide cuff on shoulder
4. Bring long strap around back and secure in front at shoulder unit
5. Adjust upper arm cuff and secure with small strap around arm
6. Adjust shoulder cap with two tabs for a snug and comfortable fit

CONTENTS

The product is provided nonsterile and is intended to be reusable on the same person multiple times. The product is manufactured out of 100% latex-free neoprene (CR). Exclusive of binding/straps/cover fabric.
Not made with Natural Rubber Latex.

CLEANING

Hand wash in warm water with a mild detergent, rinse thoroughly and drip dry. DO NOT MACHINE DRY.

STORAGE

When not in use, store in a cool, dry place.

DISPOSAL

Check product for excessive wear prior to each use. Discard the device if it shows signs of excessive wear and replace with a new product as necessary.

EN

FR

DE

IT

ES

PT

NL

BG

HR

CS

DA

ET

FI

EL

HU

LV

LT

PL

RO

SK

SL

SV

NO

INSTRUCTIONS FOR USE

463

SHOULDER WRAP

WHEN TO CONSULT A HEALTHCARE PROFESSIONAL

Consult a healthcare professional if experiencing any adverse reaction to the device including skin reaction or changes in condition.

NOTICE TO USER AND/OR PATIENT

In the event of an adverse incident related to the device, remove the device and report the incident to the manufacturer.

LABEL SYMBOLS

The symbols used on the product label or product literature are defined in the Symbols Legend here: www.usb-eifu.eu/symbols

EN

FR

DE

IT

ES

PT

NL

BG

HR

CS

DA

ET

FI

EL

HU

LV

LT

PL

RO

SK

SL

SV

NO

 United Sports Brands, Inc.
11488 Slater Ave.
Fountain Valley, CA 92708 USA
For Info:
www.McDavidUSA.com
+1-800-233-6956

 United Sports Brands
Janssen-Pharmaceuticaal
Post Box 4
2440 Geel, Belgium
For Info: www.McDavid.eu

 CH Imported by:
United Sports Brands
Switzerland GmbH
Westbahnhofstrasse 1
CH-4500 Solothurn, Switzerland
+32 (0) 14 85 47 85

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eIFU Ref: PN ART EU – MD463-IFU Rev 01 © 2024
QNET B.V.
QNET CH-REP GmbH,
QNET LTD.